

90 day limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for 90 days.

This device was built to exacting standards and carefully inspected prior to shipment.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2006 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

premium guard
static air mattress



EU Authorized Representative
Drive International, LLC
Globe Lane - Stockport
DUKINFIELD SK16 4RE, GB

www.drivemedical.com

drive
MEDICAL DESIGN & MANUFACTURING

ATTENTION!

EMERGENCY DEFLATION PROCEDURE:

MATTRESS MUST BE DEFLATED BEFORE STARTING CPR. FOR QUICK DEFLATION, COMPLETELY UNSCREW LARGE RED VALVE TURNING COUNTER CLOCK WISE.

directions for use

INFLATION:

1. From the foot end of bed, unroll the mattress on top of regular mattress with valve on left side, smooth out and place each of the 4 corner straps around standard hospital mattress.
2. Make up bed as usual with sheets, leaving mattress valve exposed. (Incontinent pads may be used).
3. Patient may now be placed on mattress.
4. Valve system:
 - a. Unscrew top cap.
 - b. Make sure the bottom valve is closed firmly.
5. Insert inflator into top of red valve and inflate for 60 seconds.

HAND CHECK:

- A. With patient in place (lying down) gently insert your hand palm up under air mattress directly beneath patients sacrum (buttocks).
- B. Always keep the back of your hand flat against the hospital mattress. If the patient's weight is directly resting on the palm of your hand, the air mattress will need to be reinflated for 30 seconds. If you cannot feel the patient, the air mattress is working efficiently and additional inflation is not required.

NOTE:

Periodically check patient's skin for redness or deterioration. Consult a physician should any occur.



REF 14425 shown