

## lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2006 Medical Depot, Inc. All rights reserved.  
Drive is a trademark of Medical Depot, Inc.  
Port Washington N.Y. 11050 USA Made in China

## deluxe electric patient lift

with rechargeable,  
removable battery

drive  
MEDICAL DESIGN & MANUFACTURING

[www.drivemedical.com](http://www.drivemedical.com)

## deluxe electric patient lift

The Drive Deluxe electric lifter is designed to allow an attendant to ergonomically transfer a patient from one resting position to another. It is not a transport device. Use a sling that is recommended by the individual's physician. During transfer, with patient suspended in a sling to the lift, do not roll caster base over objects such as raised carpet bindings, doorframes, or any uneven surfaces that would create an imbalance of the lift. Use steering handles at all times to maneuver the lift.

## installation

### Components:

- a. Mast with Boom Already Assembled included Linak Jumbo Actuator (1ea)
- b. Base (1ea)
- c. Base Spreading Adjustment Handle (1ea)
- d. LINAK JUMBO set - battery box, control box and control handset (1 ea)
- e. "C" shaped push bars (2 ea)
- f. 4/6 point cradle (1 ea)

### Procedure:

- a. Remove the screws (#16, #17) and washer (#20) from the base as indicated.
- b. With the boom pointed in the same direction as the base legs, insert the mast (#2) into the mast sleeve and ensure the mast is in place.
- c. Install the screw (#16, #17) and washer (#20) on to the mast (#2) through mast sleeve.
- d. Install the push handles (#14, #15) into the reserved position of the mast (#2) and ensure the screws (#19) are screwed tightly.

Please refer to the following pictures.



figure 1



figure 2



figure 3

- e. Unscrew the gold colored screw (#18) from mounting bracket (#9) located behind the mast (#2).
- f. Mount the Linak Jumbo Control Box (#4) on the bracket (#9) and screw back the bolts (#18) on the bottom of the bracket.
- g. Place and clip the Jumbo battery pack (#7) on the top of the bracket.
- h. Insert the base spreading adjusting handle (#3) into the socket located behind the mast sleeve.
- i. Attach the cradle (#11) to the end of the boom and make sure the bolts are screwed tightly.
- j. Now, the Lifter is ready to use.